

Ready. Set. Sleep.



This product may not be right for you. Read the warnings and instructions before use.

F&P SleepStyle +

Ready.

With simplicity, comfort and functionality woven into every detail, SleepStyle™+ gives you the freedom to sleep.





Quiet, integrated design

One piece, to minimize the space on your bedside table. Our fully integrated PAP unit includes the humidifier and power supply in a compact footprint.



User-friendly menu

The large, forward-facing, textured button works in harmony with the menu, allowing quick navigation and ease of use.*

* 16 out of 20 patients rated setting up the SleepStyle+ device as easy or very easy.



F&P SleepStyle +

Set.

Fewer buttons to push, less complexity,
smart features – in just a few steps,
you're on your way to better sleep.



Easy-access chamber

One press of a button gives you access to the water chamber, with an over-moulded watertight silicone seal.



Auto Start/Stop

Start therapy by putting on your mask and breathing in. To stop, simply remove it.*



Breathe easy

Freely breathe with our new standard filter, now with a finger tab to make it easy to detach it from the CPAP. The optional ultra-fine filter provides 99% filtration efficiency at 0.5-0.7 micron particles such as dust mites, pet dander and pollen.+


* Auto Start/Stop needs to be enabled by your Healthcare Provider or Clinician.
+ The optional ultra-fine filter incorporates a bio-based frame made from renewable materials. The ultra-fine filter frame is made with approximately 55% biorenewable plastic, which is allocated based on the amount of ISCC certified material used in its production. License code: ISCC-PLUS-Cert-DE100-11081121.



Ultra-fine Filter Frame*

Sleep.

Simplified design, advanced technology. SleepStyle+ adjusts to your needs by combining design with built-in technology that helps you sleep comfortably through the night.

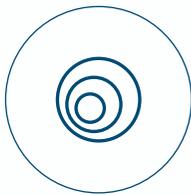

ThermoSmart™

Keep humidity at a comfortable level through the night, with our world-class humidification system. The heated breathing tube uses AirSpiral™ technology to minimize condensation.



SensAwake™

A responsive ramp all night. When the system detects wakefulness, pressure is automatically decreased to a comfortable level, then ramps up when you settle back to sleep.



Expiratory relief

Breathing feels natural, with PAP therapy pressure that automatically matches your breathing patterns. Pressure decreases when you exhale and increases as you inhale.

By using the SleepStyle App, you can access your sleep data instantly, and keep track of your PAP therapy progress. With alerts about common therapy issues and helpful advice, you're empowered to take charge of your sleep.



SleepStyle App

Use the app to see your therapy results, track progress over days, months and years, and get alerts for common therapy issues.



Scan to download



or download directly



F&P SleepStyle+ Auto/CPAP Performance Features

Performance Features	SleepStyle+ Auto
Fully integrated humidifier	✓
Auto-adjusting pressure	✓
Auto Start/Stop	✓
ThermoSmart technology**	✓
SensAwake™	✓
Expiratory relief	✓
Central apnea event detection	✓
Ramp	✓
Auto-altitude adjustment	✓
Leak compensation	✓
Oxygen compatible	✓
Efficacy reporting	✓
Compliance reporting	✓
F&P InfoUSB™	✓
Bluetooth* wireless technology	✓
Cellular modem*	✓

Compatible with	SleepStyle+ Auto
F&P InfoSmart™	✓
F&P SleepStyle App*	✓

Device Model Product Codes REF

Region	SleepStyle+ Auto
Australasia	SSPAAA

Included with the F&P SleepStyle+ Auto

- 1 x Carry Bag
- 1 x SleepStyle+ Device
- 1 x Breathing Tube**
- 1 x Power Cord
- 1 x Use and Care Guide
- 1 x Water Chamber
- 1 x Chamber Seal
- 1 x Outlet Seal
- 1 x F&P InfoUSB (already in InfoUSB port)
- 1 x Air Filter (already in device)
- 1 x Spare Air Filter
- 1 x Spare Elbow (for use with a standard breathing tube)**
- 1 x SleepStyle+ Quick Reference Guide



Prescription only. Use only as directed. Always follow the instructions for use. Your healthcare professional will advise you whether this product is suitable for you/your condition.

* Not available in all models. Cellular service is dependent on your country's network availability which may change at any time. ** A ThermoSmart Heated Breathing Tube is required to activate ThermoSmart. † Not available in all countries. †† Applied parts - to fit 22 mm (0.86 in.) Conical Connector. † F&P Solo is sold with nasal or nasal pillows cushions. These can also be bought individually and interchanged. F&P, AirSpiral, Brevida, Eson, Evora, InfoSmart, InfoUSB, SensAwake, SleepStyle, ThermoSmart, and Vitera are trademarks of Fisher & Paykel Healthcare Limited. For patent information, see www.fphcare.com/ip. Apple and the Apple Logo are registered trademarks of Apple Inc. registered in the U.S. and other countries. The Bluetooth® word mark and logos are registered trademarks owned by Bluetooth SIG Inc. and any use of such marks by Fisher & Paykel Healthcare is under license. Other trademarks and trade names are those of their respective owners. Google Play and Google Play logo are trademarks of Google Inc. 626162 D © 2023. Fisher & Paykel Healthcare Limited.